

Whip Mix FlexiMount™ Universal Mounting Support

Dynamic Stability When Mounting Casts.



- No glue, no accelerants, no waiting.
- Easily secure casts together for mounting.
- Works with any brand of articulator.



Available in the following packages:

- 20011831 FlexiMount 3 pack
- 20011832 FlexiMount 20 pack
- 20011833 FlexiMount 100 pack



RESTORATIVE ORAL HEALTH

www.whipmix.com

800-626-5651



FlexiMount Instructions

- Mount the upper cast as you would normally.
- Score the lower cast adequately and soak the base of the cast.
- Using the FlexiMount end with two holes, attach the FlexiMount device to the thumbscrew on the upper member.
- Open or invert the articulator so that you can relate the lower cast to the upper cast.
- If you are using an interocclusal record, seat it in place between the two casts.
- While holding the casts securely together, wrap the FlexiMount device around the casts and secure the other end to the thumbscrew on the upper member.
- Adjust the arms of the FlexiMount device if needed so that you have even pressure holding the casts together in good balance.
- Pull down to tighten individual arms of the device if needed for dynamic stability. This allows you to adjust the casts while keeping them in a stable position.
- Add mounting stone to the center of the cast and mounting plate.
- Once the stone has set, release the FlexiMount device from the upper member and remove the lower cast from the articulator.
- Using a sharp blade, cut the FlexiMount device away from the base of the cast.
- Add your final layer of mounting stone.

NOTE: For use with the Denar Mark 300 series articulators, use the clear button in your package in place of the thumbscrew on the upper member.

